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Manufactured by: Johnson & Johnson Vision Care, Inc. - 7500 Centurion Parkway- Jacksonville, Florida 32256 - USA

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São Paulo – SP - C.N.P.J. 54.516.661/0001-01

Resp. Tech.: Carla Carvalho de Silveira CRQ No. 04343300

Validity (year / month) and Lot: see package side

Sterile: steam sterilized product.

Contents : contact lenses

Quantity : see product packaging

Single Use and Daily Disposal

SALE UNDER REFRACTIONAL MEDICAL PRESCRIPTION

CONSULT YOUR OPHTHALMOLOGIST REGULARLY.

STORE THE PRODUCT IN ITS ORIGINAL PACKAGING AT AMBIENT TEMPERATURE.

DESTROY AFTER RECOMMENDED USE.

See Instructions for Use for Warnings, Precautions and other information.

Caption: **Packaging solution :** 1 Borate saline solution buffered with methyl cellulose ether 2 Saline borate solution buffered with povidone. **Material :** 3 The lens material contains silicone and meets Class 1 UV absorption standards with transmissibility of less than 1% UVB radiation (280-315 nm) and less than 10% UVA radiation (315-380 nm). All other ACUVUE® products meet Class 2 UV absorption standards with transmissibility of less than 5% UVB radiation and 50% UVA radiation.

Consumer Services

0800 762 5424

www.acuvue.com.br

1. PRODUCT DESCRIPTION and INTENDED USE

This leaflet refers to ACUVUE® disposable daily contact lenses that are intended to be worn for less than 24 hours while you are awake. Your vision specialist should prescribe the lenses and determine your usage plan. Lenses do not require cleaning or disinfection and must be discarded after removal.

- The spherical daily disposable contact lenses **ACUVUE®** are for daily use and correction optics of myopia (see trouble away) and hyperopia (difficulty seeing to close) in people with eyes healthy that can have up to 1:00 D (diopter) for astigmatism.
- The contact lenses the daily disposable **ACUVUE®** for ASTIGMATISM are intended for everyday use and correction optics of myopia (difficulty seeing to far) and hyperopia (difficulty seeing to close) in people with healthy eyes that may have astigmatism .
- The daily disposable contact lenses **ACUVUE®** for presbyopia are intended for everyday use for the correction optical myopia (difficulty seeing to far) and hyperopia (difficulty seeing to close) in people farsighted with healthy eyes that are up 0.75D or less (diopter) of astigmatism.

DO NOT USE YOUR LENSES WHEN YOU ARE SLEEPING.

WARNING: UV RADIATION ABSORBING CONTACT LENSES do not replace the use of other eye protection methods that absorb UV radiation, such as protective eyewear or sunglasses, as they do not fully cover the eyes and the areas around them. You should continue to wear UV radiation absorbing eye protection.

NOTE: Prolonged exposure to UV radiation is one of the risk factors associated with the development of cataracts. Exposure is based on various factors such as environmental conditions (altitude, geography, cloudy skies) and personal factors (duration and nature of outdoor activities). Radiation (UV) blocking contact lenses help protect against damage caused by UV radiation. However, clinical studies have not yet been performed to demonstrate that wearing contact lenses with a radiation (UV) blocker reduces the risk of developing cataracts or other eye diseases. Consult your ophthalmologist for more information.

Your ophthalmologist should prescribe lenses for single use and daily replacement only. Your eye care professional will determine your usage schedule (how long you should wear the lenses per day). Your lenses should be discarded after they are removed. Therefore, no cleaning or disinfection is required.

2. CONTRAINDICATIONS (in which situations you should not use)

DO NOT USE these lenses when you have any of the following conditions:

- Inflammation or infection in or around the eyes or eyelids
- Any eye disease, injury or abnormality that affects the cornea, conjunctiva or eyelids
- Any previously diagnosed problem that makes wearing contact lenses uncomfortable
- Severe dry eye
- Reduced corneal sensitivity
- Any systemic disease that can affect the eyes or worsen with contact lens use
- Allergic reactions on the surface of the eyes or in the tissues around them that can be induced by the use of contact lenses or worsen due to their use.
- Any active eye infection
- If eyes get red or irritated
- Eye irritation caused by allergic reactions to ingredients in contact lens solutions (eg lubricating drops). These solutions may contain chemicals or preservatives (such as mercury, thimerosal, etc.) to which some people may develop an allergic response .

3. WARNINGS - What You Should Know About Using Contact Lenses:

EYE PROBLEMS, INCLUDING CORNEAL ULCERS, CAN DEVELOP RAPIDLY AND LEAD TO LOSS OF VISION.

IF YOU FEEL OR PRESENT:

- Eye discomfort
- Excessive tearing
- Changes in vision
- Loss of vision
- Redness in the eyes, or
- Other eye problems

YOU MUST IMMEDIATELY REMOVE THE CONTACT LENSES AND CONTACT YOUR OPHTHMOLOGIST.

- When prescribed by your ophthalmologist for single use and daily disposal (ie, your ophthalmologist instructs you to remove and discard your lenses at the end of each day of use in accordance with the product's directions for use), you must not use the lenses during sleep. Clinical studies have shown that the

risk of serious eye problems is increased when lenses are worn while sleeping.

- Studies have shown that contact lens wearers who smoke have a higher rate of eye problems than non-smokers.

- Problems with contact lenses or lens products can result in serious damage to the eye.
- Proper use and care of contact lenses and lens products is essential to the safe use of this product.

Water Activity

Instructions for Use

Do not expose your contact lenses to water while you are wearing them.

WARNING : Water can contain microorganisms that can lead to serious infection, loss of vision, or blindness. If your lenses are submerged in water while participating in water sports or swimming in pools, bathtubs, lakes or oceans, you must dispose of them and replace them with new lenses immediately after use in these environments. Consult your ophthalmologist regarding recommendations for wearing contact lenses in any aquatic activity.

¹ New England Journal of Medicine, September 21, 1989; 321 (12), pp. 773-783

4. PRECAUTIONS

For the health of your eyes, it is important to carefully follow the handling, placement, removal and use instructions contained in this guide, as well as any extra care prescribed by your ophthalmologist (see the sections “Lenses handling and placement” and “Lens use ”).

General Precautions:

- If you wear contact lenses to correct presbyopia using monovision, you may not be able to get the best corrected visual acuity for near or far vision. Visual needs are different for different people, so your ophthalmologist should work with you to select the most appropriate type of lens for your needs.
- **Always** contact your eye care professional before using any medication in your eyes.
- **Be alert** as certain medications such as antihistamines, decongestants, diuretics, muscle relaxants, tranquilizers and those for motion sickness can cause dry eyes, increased lens perception (sensation of the presence of lenses in the eyes) or blurred vision. Always tell your eye care professional if you have problems with your lenses while using these medications. Depending on your symptoms, your ophthalmologist may recommend using moisturizing drops suitable for use with soft contact lenses, or may recommend that you stop wearing contact lenses when using these medications.
- **Be alert** as oral contraceptive users may develop changes in visual quality and comfort while wearing contact lenses.
- As with any contact lens, follow-up appointments are necessary to ensure the continued health of your eyes. Ask your ophthalmologist about the recommended follow-up schedule.
- Do not change contact lens type (eg brand, etc.) or parameters (eg diameter, base curve, lens strength, etc.) without consulting your ophthalmologist .
- Always have a working pair of glasses with a current prescription available to wear if you become unable to wear contact lenses or in circumstances where contact lens wear is not recommended.

As with any contact lens, follow-up visits are necessary to ensure continued eye health. Consult your ophthalmologist for the recommended follow-up schedule.

Who Should Know You Are Wearing Contact Lenses:

- **Inform** all your doctors that you wear contact lenses.
- **Always** inform your employer that you are a contact lens wearer. Some activities may require you to wear protective eye gear or may require you not to wear contact lenses.

5. ADVERSE REACTIONS (Side effects) – Possible problems and what to do

Possible reactions:

Be aware that reactions can occur while you are wearing contact lenses and may or may not be associated with the following symptoms:

- Burning, pinching/stinging and/or itchy eyes
- Reduced feeling of comfort with contact lenses
- Sensation of something in the eye (foreign body, scratched area)
- Swelling or inflammation in or around eyes
- Redness in the eyes
- Problems with the eyelids
- Lachrymation
- Unusual secretions in the eyes

- Reduced visual quality
- Blurred vision
- Perception of rainbows or halos around objects
- Sensitivity to light (photophobia)

When any of the above symptoms occur, a serious eye problem may be present. You should **immediately see your eye care professional** so that the problem can be identified and treated, if necessary, to avoid serious eye damage.

How to recognize problems and what to do

You should perform a simple three-part self-examination at least once a day. Ask yourself:

- How do the lenses feel in my eyes?
- How do my eyes look?
- Did I notice any changes in my vision?

If you notice changes, you must **IMMEDIATELY REMOVE YOUR LENSES**. If the change or discomfort subsides, discard the lenses and put a new pair of lenses in your eyes.

If, after inserting the new lenses, the change continues, **REMOVE THE LENSES IMMEDIATELY AND CONTACT YOUR OPHTHALMOLOGIST**.

INSTRUCTIONS FOR USE

For the health of your eyes, it is important to carefully follow the handling, placement, removal and use instructions contained in this guide, as well as the extra care prescribed by your ophthalmologist. If you are unable or unable to follow the recommended care procedures, you should not attempt to wear contact lenses.

When you buy your lenses for the first time, make sure you can put them on and off (or make sure you are advised by someone else who can take the lenses out for you properly).

Step 1: Getting Started

It is essential that you learn and have good hygiene in the care and handling of your new lenses. Cleanliness is the first and most important aspect of proper contact lens care. In particular, your hands should be clean, dry and free from any soap, lotion or cream residue before handling the lenses. Before starting:

- Always wash your hands thoroughly with a mild soap, rinse them thoroughly, and dry them with a lint-free towel before touching the lenses.
- Avoid using soaps containing moisturizing cream, lotions, or oily cosmetics before handling lenses. These substances can come into contact with the lenses and interfere with their successful use.
- It's best to put your lenses on before makeup.

Step 2: Opening the package

Always confirm that the lens parameters (eg diameter (DIA), base curve (CB), lens power (D), etc.) printed on the multipack and/or individual lens container match the prescription. **DO NOT** use them if they don't match.

Cashier

Each multipack contains individually wrapped lenses. Each lens comes in its own sealed plastic package containing saline buffered with borate and cellulose methyl ether. This package is specifically designed to keep the lenses sterile while the package is sealed. In the European Union, borates (boric acid & sodium borate) are defined as CMR 1B substances in a concentration above 0.1% weight by weight and are safe when the product is used according to label instructions.

lens packaging

To open an individual lens package (*blister*), follow these simple steps:

DO NOT use if sterile packaging is broken or damaged.

1. Shake the package and check if the lens is floating in the solution.
2. Carefully peel back the foil closure to reveal the lens. ~~Remove the laminated seal to gain access to the lens.~~ By stabilizing the lens package in a flat location, you will minimize the possibility of a solution spillage.
3. Place a finger on the lens (do not touch it with your fingernail) and slide it out of the packaging compartment until the lens is out of it.

NEVER use tweezers or other equipment to remove the lens from its housing, unless specifically indicated for such use.

Occasionally, a lens may attach to the inside surface of the laminated seal when opened, or to the plastic in the packaging itself. It will not affect the sterility of the lens. It will still be perfectly safe to use. Carefully remove the lens and inspect it following the handling instructions.

Tips for handling lenses

- Handle your lenses with your fingers and be careful to avoid contact with your fingernails. It's good to keep your nails short and filed.
- Get into the habit of always handling the same lens first to avoid mixing them up.
- Once you have removed the lens from its packaging, examine it to make sure it is moist, clean and free of any nicks or tears. DO NOT use the lens if there is any evidence of damage. Use the next lens in the box.

Step 3: Putting the lenses in the eyes

Remember to always start with the same eye.

Once you have opened the lens package, removed and examined it, follow these steps to place the lens in the eye:

1. MAKE SURE THAT THE LENS IS NOT IN ITS POSITION INVERTED, following the procedure :

Place the lens on the tip of your index finger and check it. The lens should have a natural, curved, round shape. If the lens edges tend to point outward, the lens is inverted. Another method is to lightly press the lens between your thumb and forefinger. Edges should turn inwards. If the lens is inverted, the edges will turn out slightly.

OR

Place the lens on the tip of your index finger and looking at the lens, locate the numbers 1-2-3. Numbers 1-2-3 indicate correct orientation while 1-2-3 inverted indicates the lens is inside out. If the lens is inside out (1-2-3 inverted), invert the lens and locate the numbers again to confirm correct orientation.

2. With the lens on your index finger, use your other hand to hold the upper eyelid so you don't blink.
3. Pull down your lower eyelid, using the other fingers of your hand with the lens.
4. Look up and place the lens gently on the lower white part of the eye.
5. Release your eyelid slowly and close your eye for a moment.
6. Blink several times to center the lens.
7. Use the same technique when inserting the lens into the other eye.
8. There are other methods for lens placement. If the above method is difficult for you, ask your ophthalmologist about an alternative method.

Step 4: How to check if your lenses are correctly placed

Once you have successfully inserted your lenses, ask yourself:

- Do I see well?
- How do the lenses feel in my eyes?
- Do my eyes look okay?

If after putting the lenses in, your vision is blurry, check the following:

- The lenses are not centered on the eyes (see “ *Step 5: How to Center the Lenses*” later in this instruction for use).
- If the lens is centered, remove it (see “Removing Your Lenses”) and check the following points:
 - Cosmetics or oil on the lens. Discard the lens and insert a new lens.
 - The lens is in the wrong eye.
 - The lens is inverted (also not comfortable as it should be). See “Step 3: Putting the lens in your eyes”.

If you detect that your vision is still blurry after checking the above possibilities, remove both lenses and see your eye care professional.

Note: If a lens is noticeably uncomfortable when putting it on or becomes uncomfortable soon after insertion, remove the lens immediately and contact your eye care professional. If your eye and lens self- examination shows any other problems, IMMEDIATELY REMOVE THE LENSES AND CONTACT YOUR OPHTHALMOLOGIST.

Step 5: How to center the lenses

A lens, positioned on the cornea (center of the eye), very rarely moves to the white part of the eye during use. However, this can occur if the insertion and removal procedures are not carried out properly. To center a lens, do one of the following:

- Close your eyelids and lightly massage the lens into place through the closed eyelids.

OR

- Gently move the lens off-center to the cornea (center of the eye) while the eye is open, using the upper or lower lid edge to move the lens into the correct position.

When wearing lenses, remember the following important precautions:

risk conditions

- If you use aerosol (*spray*) products such as *hairspray* , when wearing lenses, keep your eyes closed until the *spray* dissolves in the air.
- Avoid exposing yourself to any harmful or irritating vapors, as well as smoke, when wearing lenses.
 - Never rinse your lenses with tap water. Tap water contains many impurities that can contaminate or damage your lenses and cause eye infections or damage .

water activity

- Do not expose your contact lenses to water while wearing them.

How to lubricate/apply solutions

- Your eye care professional may recommend an appropriate lubricating/humectant solution to moisten (lubricate) your eyes when you are wearing contact lenses.
- **Do not use saliva** or anything other than the recommended solution to lubricate your lenses. Do not put lenses in your mouth.

sharing lenses

Never share your lenses with another person. They have been prescribed for your eyes and to correct your vision as needed. Sharing lenses greatly increases the chance of eye infections.

Adherence to prescribed replacement and usage schedule

- **Never** use your lenses beyond the time indicated in the product's instructions for use or without the extra care recommended by your ophthalmologist.
- **Always** dispose of used lenses as indicated in the instructions for use and according to the extra care prescribed by your ophthalmologist.

REMOVING YOUR LENSES

CAUTION: Always make sure the lens is in the cornea (in the center of your eye) before attempting to remove it. Confirm this by covering the other eye. If vision is blurry, the lens is in the white part of the eye or not in the eye. To locate the lens, inspect the upper eye area, looking down in the mirror while lifting the upper eyelid. Then inspect the lower area by pulling the lower eyelid down.

1. Wash, rinse and dry your hands thoroughly. You should follow the method recommended by your ophthalmologist. Here is an example method. Clamping method:

Step 1. Look up, slide the lens to the bottom of the eye using your index finger.

Step 2. Gently pinch the lens between your thumb and forefinger.

Step 3. Remove the lens.

2. Follow the instructions in the next section, "Caring for your lenses".

Note : For the health of your eyes, it is important that the lenses move freely in your eye. If the lens sticks to the eye (stops moving), apply a few drops of the recommended lubricating solution. Wait until the lens starts to move freely in the eye before removing it. If the lens remains stationary, consult your ophthalmologist immediately.

If any chemical substance (household products, gardening solutions, laboratory chemicals, etc.) spills into your eyes: **IMMEDIATELY FLUSH YOUR EYES WITH RUNNING WATER AND QUICKLY CONTACT YOUR OPHTHALMOLOGIST DOCTOR OR REFER TO AN EMERGENCY ASSISTANCE AS SOON POSSIBLE).**

6. NOTIFICATION OF ADVERSE REACTIONS (Side effects)

Any incident while wearing ACUVUE® contact lenses should be reported to the manufacturer and/or its authorized representative and/or regulatory authority in your country.

SYMBOLS

The following symbols may appear on the product label or box:

Symbol	Definition
	Caution, see instructions for use
	Made by or in
	Manufacturing date
	Product Expiration/Valid Date
	Product manufacturing batch number
	Steam Sterilization
	Do not reuse (Single use)
	Do not use if packaging is violated
	Regarding disposable material
	ATTENTION: sale under medical prescription
	Medical device in the European Community
	Indicates a unique sterile barrier system
	UV filter
	Authorized Representative in the European Union
	CE mark and notified body identification number
DIA	Diameter
BC	base curve
D	Diopter (lens degree)
CYL	Cylinder
AXIS	Axle
MAX ADD	Maximum addition that can be corrected
LOW/ L	Addition of close "low"
MID/ M	Addition of close "average"
HGH/ H	Addition of close " high "
	Correct lens orientation
	Incorrect lens orientation (lens inside out)
	Opening the package
	Contains Hazardous Substances