

Personalised Face Mask – Washable Fabric

Your three layered poly-cotton mask (with middle non-woven filtration layer) and flexible metal nose bridge has an expandable concertina design for a comfortable, close fit around your nose, mouth and chin. Adjustable elasticated ear loops fit a range of head sizes (optional silicone ear sliders and fitting tool supplied). Handmade in Europe.

Mask is available in two sizes, Standard (85 x 180mm when folded; expanded height is 170mm) and Small (70 x 160mm when folded; expanded height is 150mm).

OUTER / PRINTED FABRIC LAYER: 100% polyester 240 gr. The first protection barrier, which is moisture resistant due to its high thread density as well as easy to wash and dry.

MIDDLE FABRIC LAYER: Non-woven viscose spunlace 44gsm. Second filter barrier layer, which provides high humidity absorption as well as supporting layer separation for comfortable use.

INNER / SKIN CONTACT FABRIC LAYER: Woven Flat polyester cotton 144gsm fine thread. Third protection barrier layer, which offers high breathability due to its composition. High thread count and fabric type is comfortable and does not irritate your skin.

Important: Our masks are designed for personal use and are not suitable for medical use or as personal protective equipment (PPE). The barrier mask is not a replacement for other protective and social distancing measures. Please see your local regulations for relevant health instructions. Not suitable for medical workers, young children or those with ink allergies. Ear loops contain latex. Photobox makes no warranties, either express or implied, that this mask prevents infection or the transmission of viruses or diseases.

Wear and care:

- Wash your hands with soap and water or use hand sanitiser before handling the mask.
- Put the mask on so that it covers your mouth, nose and chin, put the elastic over your ears, adjust nose band if necessary.
- Do not touch the mask while wearing it and remove it without touching the front.
- As soon as the mask is damp from use, replace it with a clean and dry one.
- Fit and adjust ear sliders if needed (see website for fitting instructions).
- Use under adult supervision only.
- Not suitable for dry cleaning.
- THIS IS NOT A TOY. Not suitable for children under the age of 6.
- Do not use fabric softener.
- 90% filtration efficiency to particles around 3 (+/- 0,5) μm .
- Clean the face mask after each use.
- Stop using this product at the first signs of damage.

KEEP AWAY FROM FIRE CONTAINS SMALL PARTS



Wash your hands



Put on your mask



Place the loops
over your ears



Don't touch
the front



Change when
damp from use

Washing instructions:

- Machine washable with similar colours up to 60°C or hand wash for at least 20 seconds.
- Wash with your usual detergent.
- Washing your mask in a mesh wash bag can stop ear loops from getting tangled, caught or stretched.
- Air dry recommended or tumble-dry on low heat.
- Your mask can be ironed on a medium setting.
- Make sure your mask is completely dry before using.

How to fit ear adjusters to your mask:

- Fold the wire over the elasticated ear loop.
- Push the two wire ends into the rubber adjuster's larger opening.
- Slide the adjuster down, over the wire and onto the elastic.
- Remove the wire.
- Repeat for the second ear loop.
- Use the rubber adjuster to tighten the ear loops so your mask fits comfortably on your face.

PH*TOBOX

© Photobox 2020, registered company address is Herbal House, 10 Back Hill, London EC1R 5EN. Please note: this is our registered company address, and should not be used for returns or customer support enquiries.