IFU000796E
02/2025

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PLEASE READ CAREFULLY AND RETAIN FOR FUTURE REFERENCE. ACUVUE CONTACT LENSES (CL) ARE MEDICAL DEVICES AND SHOULD BE FITTED BY AN EYE CARE PROFESSIONAL (ECP). ALWAYS FOLLOW YOUR ECP'S DIRECTIONS AND THE INSTRUCTIONS IN THIS LEAFLET.

PRODUCT DESCRIPTION and INTENDED USE: Not all of the listed products might be available in your country.

ACUVUE OASYS Brand Contact Lenses with HydraLuxe; ACUVUE OASYS Brand Contact Lenses for ASTIGMATISM with HydraLuxe; ACUVUE OASYS MAX 1-Day Contact Lenses, ACUVUE OASYS MAX 1-Day Contact Lenses for ASTIGMATISM; ACUVUE OASYS MAX 1-Day MULTIFOCAL Contact Lenses; ACUVUE OASYS MAX 1-Day MULTIFOCAL Contact Lenses for ASTIGMATISM

Packing Solution: Borate buffered saline with methyl ether cellulose.

Material: senofilcon A

1-DAY ACUVUE MOIST Brand Contact Lenses;

1-DAY ACUVUE MOIST Brand Contact Lenses for ASTIGMATISM; 1-DAY ACUVUE MOIST MULTIFOCAL Brand Contact Lenses.

Packing Solution: Borate buffered saline with povidone.

Material: etafilcon A.

Borates (boric acid & sodium borate) are defined as CMR 1B substances in a concentration above 0.1% weight by weight and are safe when the product is used according to label instructions. Daily Disposable ACUVUE contact lenses that are intended to be worn for < 24 hours while awake. Your Eye Care Professional should prescribe the lenses and determine your wearing schedule. Your lenses do not require cleaning or disinfection and should be discarded upon removal.

•Daily Disposable ACUVUE Spherical Contact Lenses are intended for Daily Wear for the optical correction of myopia (short-sightedness) and hyperopia (long-sightedness) in persons with healthy eyes that may have $\leq 1.00D$ of astigmatism.

•Daily Disposable ACUVUE Contact Lenses for ASTIGMATISM are intended for Daily Wear and for the optical correction of myopia (short-sightedness) and hyperopia (long-sightedness) in persons with healthy eyes that have astigmatism.

•Daily Disposable ACUVUE Contact Lenses for PRESBYOPIA are intended for Daily Wear for the optical correction of myopia (short-sightedness) and hyperopia (long-sightedness) in presbyopic persons with healthy eyes who have 0.75D or less of astigmatism.

•Daily Disposable ACUVUE Contact Lenses for PRESBYOPIA and ASTIGMATISM are intended for Daily Wear for the optical correction of myopia (short-sightedness) and hyperopia (long-sightedness) in presbyopic persons with healthy eyes who have 0.75D to 1.75D of astigmatism.

WARNING: UV absorbing CL are not substitutes for UV protection. You should continue to use UV absorbing eyewear as directed.

CONTRAINDICATIONS:

When wearing CL for vision correction, DO NOT USE CL in case of eye disease, injury, inflammation, infection, or abnormality.

•Any condition or disease of the eye or surrounding tissues induced or aggravated by wearing CL or use of CL solutions.
•If eyes become red or irritated. Some people may develop an allergic response to chemicals or preservatives in contact lens care solutions.

WARNINGS:

•Failure to adhere to these warnings can lead to severe infection, vision loss or blindness. EYE PROBLEMS (INCLUDING CORNEAL ULCERS) CAN DEVELOP RAPIDLY AND LEAD TO LOSS OF VISION. IF YOU EXPERIENCE ANY OF THE FOLLOWING SYMPTOMS (Eye Discomfort, Loss of Vision, Vision Changes, Excessive Tearing, Eye Redness), you should IMMEDIATELY REMOVE YOUR CL AND PROMPTLY CONTACT YOUR ECP.

- Do not wear CL if there is any eye disease.
- DO NOT WEAR YOUR CL WHEN SLEEPING
- DO NOT EXPOSE CL TO WATER, AS THIS MAY LEAD TO SERIOUS INFECTION. Discard and replace any CL exposed to water. (Do not wear CL during water sport).
- NEVER rinse your lenses in water from the tap

PRECAUTIONS:

DO NOT USE YOUR LENSES IF THE STERILE BLISTER PACKAGE IS OPENED, DAMAGED OR AFTER THE EXPIRY DATE.

• Only wear your own prescribed CL. Do not change your lens type or parameters without consulting your ECP. Ask your ECP about the follow-up visit schedule.

•ALWAYS CONTACT ECP BEFORE USING ANY MEDICINE IN YOUR EYES.

•NEVER use tweezers or other tools to remove your lenses from the lens container.

•Smoking and/or taking some medicines (like contraceptives or other medicines) may increase eye problems or eye discomfort.

•AVOID all harmful or irritating vapors/fumes. Keep your eyes closed during usage of aerosol products.

ADVERSE REACTIONS:

- Burning, stinging, itchy, and/or dry eyes
- Reduced lens comfort or feeling of something in your eye
- Swelling or inflammation in or around the eyes
- Eye redness
- Eyelid problems
- Watery eyes and/or unusual eye secretions
- Poor or blurred vision
- Rainbows or halos around objects
- Sensitivity to light
- Temporary impairment due to peripheral infiltrates, peripheral corneal ulcers, and corneal erosion.

If you notice any problems, you should IMMEDIATELY REMOVE YOUR CL and contact your ECP.

DIRECTIONS FOR USE:

Not following directions for use may result in severe eye problems.

•ALWAYS WASH YOUR HANDS BEFORE TOUCHING YOUR CL as CL are very sensitive to contamination.

•NEVER PUT YOUR LENSES IN YOUR MOUTH OR PUT SALIVA ON YOUR LENSES.

•NEVER ALLOW ANYONE ELSE TO WEAR YOUR LENSES.

•Follow your ECP instructions regarding wear and removal of your CL.

Lens Package: Carefully peel back the foil closure to reveal the lens.

Lens Insertion: Ensure the lens is not inside out. Place on your index finger to verify the shape. The lens should assume a natural, curved, bowl-like shape (Fig. 1). With the lens on your index finger, use your other hand to hold your upper eyelid so you will not blink. Place the lens onto the eye. (Fig 2 & 3).


Lens Removal: Look up, slide the lens to the lower part of the eye using the forefinger. Gently pinch the lens between the thumb and forefinger to remove the lens. (Fig. 4)

EMERGENCIES: If chemicals of any kind (household products, gardening solutions, laboratory chemicals, etc.) are splashed into your eyes: FLUSH EYES IMMEDIATELY WITH TAP WATER AND IMMEDIATELY CONTACT YOUR ECP OR VISIT A HOSPITAL EMERGENCY ROOM RIGHT AWAY.

REPORTING OF ADVERSE REACTIONS (Side Effects):

Any incident experienced whilst wearing ACUVUE Brand CL should be reported to the manufacturer and/or its authorized representative and/or to your national authority.

The following symbols may appear on the labels or packaging

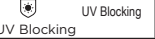Symbol	Definition
	Date of Manufacture
	Manufacture
	Use-by Date (expiration date)
	Batch Code
	Do Not Use if Package is Damaged
	Consult Instructions for Use or Consult Electronic Instructions for use
	Caution
	Sterilized Using Steam Heat
	Do Not Re-Use (Single Use)
	Medical Device in the European Community
	Indicates a Single Sterile Barrier System
	CAUTION: U.S. Federal law restricts this device to sale by or on the order of a licensed practitioner
	UV Blocking
	UV Blocking
	Authorized Representative in the European Community
	CE-mark and Identification Number of Notified Body
	Diameter
	Base Curve
	Diopter (lens power)
	Contains Hazardous Substances
	Importer