IFU000797E
02/2025

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EN Instructions for Use Daily Wear and Extended Wear ACUVUE Contact Lenses.

PLEASE READ CAREFULLY AND RETAIN FOR FUTURE REFERENCE. ACUVUE CONTACT LENSES (CL) ARE MEDICAL DEVICES AND SHOULD BE FITTED BY AN EYE CARE PROFESSIONAL (ECP). ALWAYS FOLLOW YOUR ECP'S DIRECTIONS AND THE INSTRUCTIONS IN THIS LEAFLET.

PRODUCT DESCRIPTION and INTENDED USE:

Senofilcon A Contact lenses (38% H₂O) in borate buffered saline with methyl ether cellulose: Intended for Daily Wear (14 Days - Frequent replacement) (less than 24 hours, while awake) or Extended Wear (may be used continuously for up to 7 days/6 nights and discarded upon removal) (greater than 24 hours, including while asleep). When used for Extended Wear, no cleaning or disinfection is required. Your ECP will determine your wearing schedule and your replacement schedule.

ACUVUE OASYS Brand Contact Lenses with HYDRACLEAR PLUS are intended for the optical correction of myopia (short-sightedness) and hyperopia (long-sightedness) in persons with healthy eyes who may have 1.00D or less of astigmatism. Lenses are also indicated for therapeutic use as a bandage lens for certain ocular conditions and to relieve corneal discomfort. Your ECP will tell you if you have such a condition and may prescribe additional medications or replacement schedules for your individual condition. Use as a bandage lens should be closely monitored by your ECP. When directed, only the ECP will apply and remove the CL. Do not self-treat any condition with a CL or eye medications without first consulting your ECP.

ACUVUE OASYS Brand Contact Lenses for ASTIGMATISM are intended for the optical correction of myopia (short-sightedness) and hyperopia (long-sightedness) in persons with healthy eyes who have astigmatism.

ACUVUE OASYS MULTIFOCAL Contact Lenses are indicated for the correction of vision in presbyopic people with healthy eyes who are presbyopic and may also need correction of myopia (short-sightedness) or hyperopia (long-sightedness) and may have 0.75D or less of astigmatism.

Senofilcon C Contact lenses (41% H₂O) in borate buffered saline with methyl ether cellulose: Intended for Daily Wear (1 month - Frequent Replacement)

ACUVUE VITA Brand Contact Lenses are intended for the optical correction of myopia (short-sightedness) and hyperopia (long-sightedness) in persons with healthy eyes who may have 1.00D or less of astigmatism.

ACUVUE VITA Brand Contact Lenses for ASTIGMATISM are intended for the optical correction of myopia (short-sightedness) and hyperopia (long-sightedness) in persons with healthy eyes who may have astigmatism.

Borates (boric acid & sodium borate) are defined as CMR 1B substances in a concentration above 0.1% weight by weight and are safe when the product is used according to label instructions.

WARNING: UV absorbing CL are not substitutes for UV protection. You should continue to use UV absorbing eyewear as directed.

CONTRAINDICATIONS: When wearing CL for vision correction, DO NOT USE CL in case of eye disease, injury, inflammation, infection, or abnormality.

- Any condition or disease of the eye or surrounding tissues induced or aggravated by wearing CL or use of CL solutions.
- If eyes become red or irritated. Some people may develop an allergic response to chemicals or preservatives in contact lens care solutions.

WARNINGS: Failure to adhere to these warnings can lead to severe infection, vision loss or blindness. EYE PROBLEMS (INCLUDING CORNEAL ULCERS) CAN DEVELOP RAPIDLY AND LEAD TO LOSS OF VISION. IF YOU EXPERIENCE ANY OF THE FOLLOWING SYMPTOMS (Eye Discomfort, Loss of Vision, Vision Changes, Excessive Tearing, Eye Redness), you should IMMEDIATELY REMOVE YOUR CL AND PROMPTLY CONTACT YOUR ECP.

- Do not wear CL if there is any eye disease.
- Do not wear your CL when sleeping except if your ECP prescribed you CL for Extended Wear (7 days/6 nights)
- DO NOT EXPOSE CL TO WATER, THAT MAY LEAD TO SERIOUS INFECTION. Discard and replace any CL exposed to water (Do not wear CL during water sport.)
- NEVER use WATER, SALINE SOLUTION, OR REWETTING DROPS TO CLEAN OR DISINFECT CL.

PRECAUTIONS: DO NOT USE YOUR LENSES IF THE STERILE BLISTER PACKAGE IS OPENED, DAMAGED OR AFTER THE EXPIRY DATE.

- Only wear your own prescribed CL. Do not change your lens type or parameters without consulting your ECP. Ask your ECP about the follow-up visit schedule.
- ALWAYS CONTACT ECP BEFORE USING ANY MEDICINE IN YOUR EYES.
- NEVER use tweezers or other tools to remove your lenses from the lens container.
- Smoking and/or taking some medicines (like contraceptives or other medicines) may increase eye problems or eye discomfort.
- AVOID all harmful or irritating vapors/fumes. Keep your eyes closed during usage of aerosol products.

ADVERSE REACTIONS: Burning, stinging, itchy, and/or dry eyes - Reduced lens comfort or feeling of something in your eye - Swelling or inflammation in or around the eyes - Eye redness - Eyelid problems - Watery eyes and/or unusual eye secretions - Poor or blurred vision - Rainbows or halos around objects - Sensitivity to light - temporary impairment due to peripheral infiltrates, peripheral corneal ulcers, and corneal erosion. If you notice any problems, you should IMMEDIATELY REMOVE YOUR CL and contact your ECP.

DIRECTIONS FOR USE: Not following directions for use may result in severe eye problems.

- ALWAYS WASH YOUR HANDS BEFORE TOUCHING YOUR CL as the CL and CL case are very sensitive to contamination.
- NEVER PUT YOUR LENSES IN YOUR MOUTH OR PUT SALIVA ON YOUR LENSES.
- NEVER ALLOW ANYONE ELSE TO WEAR YOUR LENSES.
- Follow your ECP instructions for wear, removal, storage, and case maintenance.

Lens Package: Carefully peel back the foil closure to reveal the lens.

Lens Insertion: Ensure the lens is not inside out. Place on your index finger to verify the shape. The lens should assume a natural, curved, bowl-like shape (Fig. 1). With the lens on your index finger, use your other hand to hold your upper eyelid so you will not blink. Place the lens onto the eye. (Fig 2 &3).

Lens Removal: Look up, slide the lens to the lower part of the eye using the forefinger. Gently pinch the lens between the thumb and forefinger to remove the lens.

(Fig. 4)

Caring for CL: USE ONLY RECOMMENDED SOLUTIONS TO CLEAN, DISINFECT AND STORE YOUR CL and follow the instructions provided. DO NOT heat the disinfection solution and CL. Always keep CL completely covered by disinfecting solution when stored in a closed case. If CL becomes dried out, discard it and replace it. Always empty/clean & rinse CL case using fresh, sterile disinfecting solutions/CL cleaner and wiping the lens case with a fresh, clean tissue. Replace CL case according to instructions to prevent microbial growth. For hydrogen peroxide lens care systems, use ONLY the lens case provided. Failure to neutralize hydrogen peroxide will result in severe stinging, burning and injury to the eye. EMERGENCIES: If chemicals of any kind (household products, gardening solutions, laboratory chemicals, etc.) are splashed into your eyes: FLUSH EYES IMMEDIATELY WITH TAP WATER AND IMMEDIATELY CONTACT YOUR ECP OR VISIT A HOSPITAL EMERGENCY ROOM RIGHT AWAY.

REPORTING OF ADVERSE REACTIONS (Side Effects): Any incident experienced whilst wearing ACUVUE Brand CL should be reported to the manufacturer and/or its authorized representative and/or to your national authority.

The following symbols may appear on the labels or packaging:

Symbol	Definition
	Caution, Consult instructions for use
	Manufacturer
	Date of Manufacture
	Use-by Date (expiration date)
	Batch Code
	Sterilized Using Steam Heat
	Do Not Use if Package is Damaged
	Fee Paid for Waste Management
	CAUTION: U.S. Federal law restricts this device to sale by or on the order of a licensed practitioner
	Medical Device in the European Community
	Indicates a Single Sterile Barrier System
	UV Blocking
	Authorized Representative in the European Community
	CE-mark and Identification Number of Notified Body
	Diameter
	Base Curve
	Dioptre (lens power)
	Cylinder Power
	Axis
	Highest Near Addition That Can Be Corrected
	"Low" near ADD
	"Medium" near ADD
	"High" near ADD
	Lens Orientation Correct
	Lens Orientation incorrect (Lens Inside Out)
	"Identification mark" for paper containers and wrapping
	"Identification mark" for composite materials
	Contains Hazardous Substances
	Importer in European Community